




Instructions for user of  
**Comfort Grab Rail**  
**ASSIST**

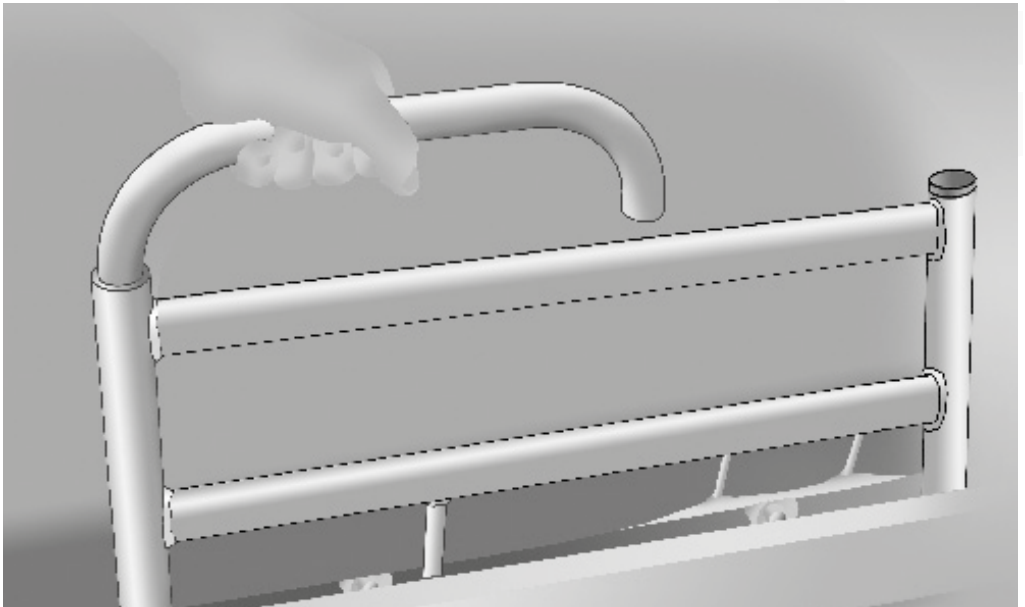

The grab rail fulfils the applicable parts of the requirements in the bed standard EN 1970:2000 and it is designed to be used as a handicap aid, for home care or similar. The grab rail is intended for individuals older than 12 years of age.



*Read the instructions  
before using the bed!*



**ComfortSystem**

*– our experience is your guarantee*






# 1. Assembly Instructions

## Comfort Grab Rail ASSIST

### 1.1 Installation/Removal

The ASSIST grab rail is installed on the sprung base's longitudinal metal profile and it can be mounted on the right or left side of the bed. The upper section of the grab rail can be angled out in various positions for use as a raising aid.

 When installing the ASSIST grab rail, the upper side mattress stop is removed.

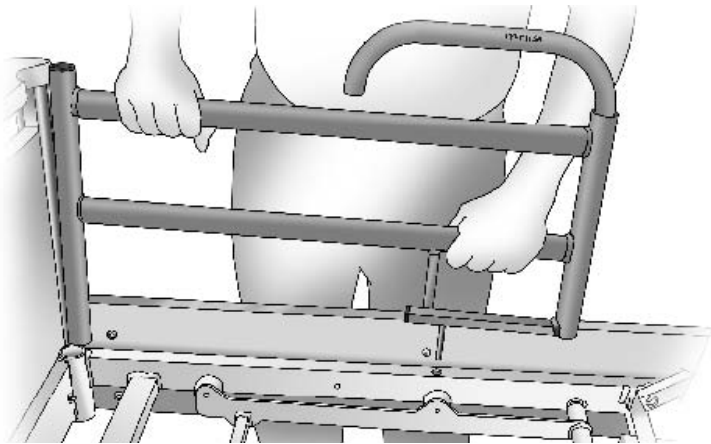


Figure 1

1. Place the grab rail, as shown in **figure 1** (the grab rail's tube has to sit in the sprung base's socket for the lift frame). The grab rail's bolt (adjustable laterally) has to go through the hole in the sprung base's longitudinal metal profile.



2. Screw the grab rail firmly into place using the enclosed locking lever as shown in **figure 2**.
3. When dismantling – do the above in reverse order

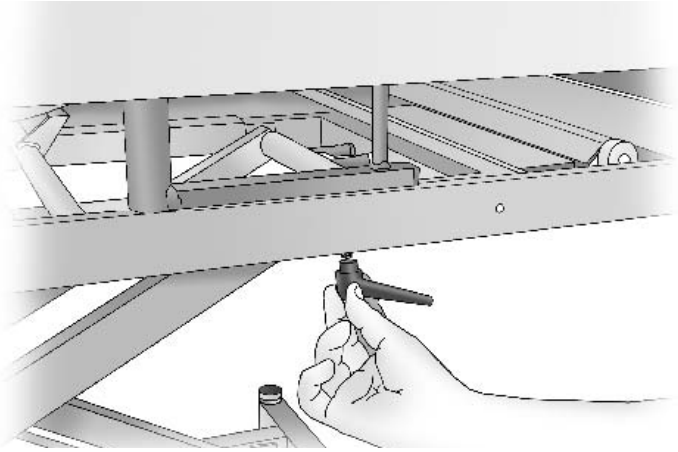


Figure 2



*Check that the grab rail sits firmly.*



## 2. Use

### 2.1 Adjusting the raising aid

The upper section of the ASSIST can be used as a raising aid and angled out in different fixed positions.

1. Lift the handle straight up as shown in **figure 3**.
2. Turn the handle to the desired position (**figure 4**) and release down. The handle 'searches' for the nearest fixed position.



Figure 3

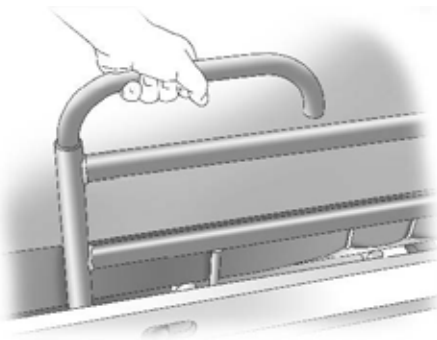





Figure 4



 The handle must not be angled into the bed or in parallel with bed (down towards the foot end)!

 The maximum load allowed on the aid is 75 kg!

 Check that the handle has found a correct position by trying to turn it.

NOTE! When used for moving to/from bed and when the raising aid is angled out from the bed, the aid must be used as shown in **figure 5**.

Do NOT use as shown in **figure 6**.



Figure 5

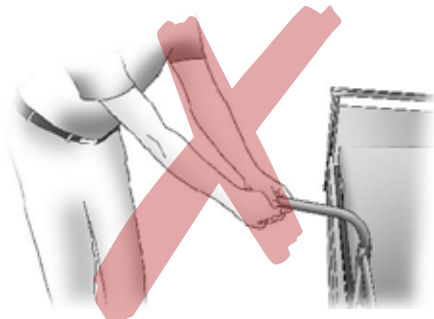


Figure 6



## 3. Care Instructions

The ASSIST grab rail does not normally require regular maintenance.

### **2.1 Normal cleaning**

The grab rail can be cleaned with readily available cleansers and disinfectants. Apply with a damp cloth.

### **2.2 Regular inspection**

Check regularly that the grab rail sits firmly on the sprung base and that the raising aid does not have an unnaturally large gap or malfunction.

## 4. Field of application

It is intended to be used as an aid when changing position in bed or when moving to/from bed.

The ASSIST grab rail may only be installed in Comfort Bed System beds manufactured after 1995.





## Our experience - your guarantee

Since its start in 1987, Comfort System has kept more or less the same ownership and has operated under the same name – a fact that is almost unique today in the branch. Our long experience and knowledge of our product assortment, means that as a customer you can feel secure in your choice of supplier.



**ComfortSystem**

*– our experience is your guarantee*