



ComfortSystem – our experience is your guarantee

# Instructions for users of Comfort Raising Aid RESOLUT



Read the instructions  
before using the bed!



## ComfortSystem

– our experience is your guarantee

CE



# 1. Assembly Instructions Comfort Raising Aid RESOLUT

## 1.1 Assembly

Before assembly:



The raising aid has to be raised!

The RESOLUT raising aid is placed in one of the sprung base's four round sockets in the corners of the bed.

- Place the raising aid in the socket as shown in **figures 1 and 2**. The track in the bottom of the lower tubular section has to fit into the socket's diagonal strut.
- The upper section of the raising aid has to 'point' out from the bed's long side. **Figure 2**.



Figure 1



Figure 2




Check that the raising aid is sitting correctly by trying to turn it round in the socket – it should NOT be possible.



## 1.2 Removal

- Lift the raising aid straight up, out of the socket. **Figure 3.**

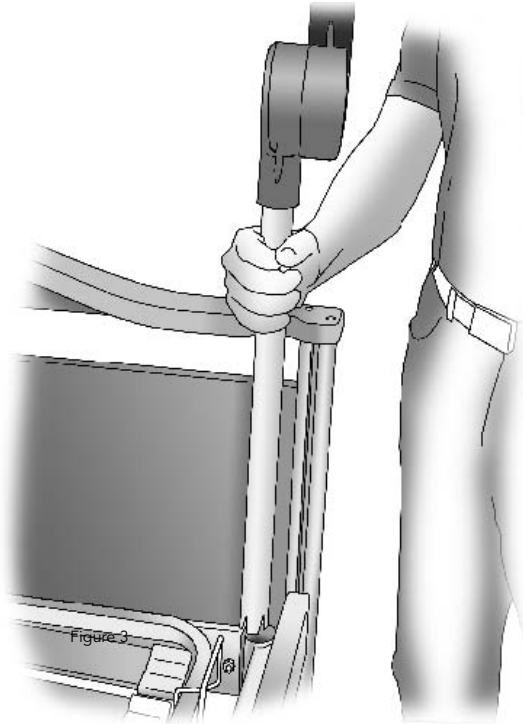

## 1.3 Use

- The RESOLUT raising aid is intended to be used as an aid when moving to/from the bed. When it is not in use, it can be lifted into the vertical position. The mechanism for locking it in different positions is in the aid's joint (the black round part in the middle!).
- Fold the upper section of the raising aid up or down by taking a hold as shown in **Figure 4.**



In the lowered position, the aid stops in the horizontal position. When lifting up – raise the aid straight up, until it ‘sticks’ in the vertical position.

NOTE! During use for moving to/from the bed, the aid has to be used as shown in **figure 5**.

Do not use as shown in **figure 6**!



The maximum load allowed on the aid is 75 kg!



Figure 4



Figure 5

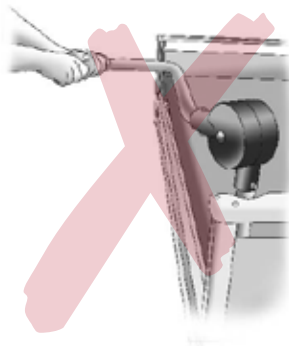


Figure 6



## 2. Care Instructions

The RESOLUT raising aid does not normally require regular maintenance.

### **2.1 Normal cleaning**

The RESOLUT raising aid can be cleaned with readily available cleansers and disinfectants. Apply with a damp cloth.

### **2.2 Regular inspection**

Check the raising aid's attachment regularly and that there isn't an unnaturally large gap in the joint.

### **2.3 Load**

- Load 75 kg (not user weight)

## 3. Field of application

To be used as an aid when moving to/from bed.

The RESOLUT raising aid may only be installed in Comfort Bed System beds manufactured after 1995.



ComfortSystem – *our experience is your guarantee*

## Our experience - your guarantee

Since its start in 1987, Comfort System has kept more or less the same ownership and has operated under the same name – a fact that is almost unique today in the branch. Our long experience and knowledge of our product assortment, means that as a customer you can feel secure in your choice of supplier.



**ComfortSystem**

– *our experience is your guarantee*

Comfort System Scandinavia AB • Box 4034 • 561 04 Huskvarna  
Tel 036-515 90 • Fax 036-500 01 • [info@Comfortsystem.se](mailto:info@Comfortsystem.se)